

From UK- Completely In the Ear Digital Hearing Aid 3 Months Batteries (99 GBP)





South East, Surrey Location https://www.freeadsz.co.uk/x-553804-z



£99 for One OR £184 for Two - Save £thousands£

• The design of this hearing aid is inspired by nature. This hearing aid will particularly suit people with slight to moderate hearing loss. People who wear glasses and also have Behind the Ear hearing aids will especially benefit from an aid completely in the ear canal. People who do not wish to advertise that they are wearing hearing aids will like this aid, which cannot be seen unless looking straight into the ear. • Red for the right ear and blue for the left ear, means you can adjust and retain the settings for your right or left ear. Each hearing aid can be adjusted for volume sensitivity to suit exactly the requirements for each ear. Special price for two hearing aids. Or buy two for the Special Price and keep one as a spare. • Each hearing aid comes with 4 sizes of ergonomically designed silicone ear plugs that fit into the ear canal so that you can try different sizes and select the cap that is correct and most comfortable. The cap should be kept clean and a brush is provided for this purpose. The silicone cap should be replaced every few months and are available from us. • As the aid fits into your ear it has a nylon wire to make it easy to remove the aid. This aid is a joy to wear - you will hardly notice it but will be able to hear what people are saying. Finally, you can again actively participate in conversations! The sound processor amplifies the volume of noise of any kind and is individually adjustable according to your needs. • The volume control is easy to adjust. Sound receiver volume is adjusted by removing the aid from your ear and using the screwdriver provided to gently reduce the volume to the minimum and then gradually increase the volume. Wear it for a day at the lower volume before turning the volume up very slightly and repeating the process. Try different settings until you find the most comfortable. When the volume is too high the aid will tend to whistle. Note that the hearing aid must NOT be worn when swimming or showering as it must











Months From ompletely



https://www.freeadsz.co.uk/x-5538 04-z Digital 닺 Completely Б



mistreatment, within one year from purchase, please return it with proof of purchase and you will receive

a brand new replacement at occurrence to woursel of Shipped ree to all UK addresses. • My Story: I baid cg1500 for an in-ear nearing and a few years ago After 2 years I accordently disposed it into a hang-basinor water. Str was out of warrant you and unrepairable. A very expensive rearing and Them the NHS stopplied to the water. Street was a street water. Street was supplied to the water of the water. Street was supplied to the water of the wat Aid

behind the ear hearing aid, but as I wear glasses I found that the frame and the hearing aid behind my ear constantly clashed and required adjustment. Also the tube from hearing aid to ear canal was uncomfortable on my ear and also very unsightly and my wife said it made me look older. Also, there was an echo.NOW, I wear a red in-ear mini in my right ear and the sound quality is excellent and I hardly know I am wearing a hearing aid.....and most people I speak with do not know either. BEST of ALL, my wife has stopped complaining that I have misunderstood what she has said or that she has to repeat herself or shout at me, because I am happy to wear the aid from morning to night. • HOW TO SAVE A FORTUNE: NHS hearing tests are free - if you think you have hearing loss in one or both ears ask your GP to arrange a hearing test. Then, you can accept the behind the ear hearing aids provided by the NHS or if you want in in-ear, less visible aid that will not clash with your glasses or sun-glasses purchase your new in-ear aids from us. The price at Boots, Specsavers, Hidden Hearing and other high-street providers could be £1000 or more. Technical Parameters: • The hearing aid is built to EU standards and bears the CE mark and also is certified that it contains no dangerous materials and has the ROHS mark. MAX-SSPL90: 118+3 dBSPL• HFA-SSPL90: 110±4dBSPL• Peak Gain: 33dB• HFA Full-on Gain: 30±5dB• Frequency Range: 200-4000HZ• THD: ?10%• Eq.Input Noise Level: 28dBSPL• Battery Current Drain: 0.6mAEach Hearing Aid Package includes:1 x hearing aid4 x Different size earplugs1 x screwdriver1 x cleaning brush1 user instruction leaflet4 packs of 6 A10 Rayovac Hearing Aid